



Scoot-a-bout

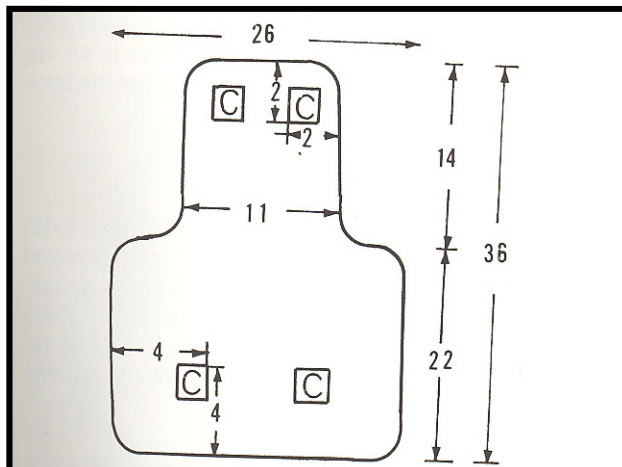


Diagram for construction of Scoot-A-Bout and placement of casters, C, and straps.



Underside of Scoot-A-Bout, showing attachment of straps and casters.

Directions:

1. Use 5-ply plywood.
2. Overall measurements are 36 x 27 inches
3. Attach four 2-inch hard rubber casters
4. Attach three adjustable straps across the bottom of board, at the two lower corners, to hold the legs, and at the center where the board narrows for the chest, to hold the trunk in position.

Idea from Functional Aids for the Multiply Handicapped edited by Isabel P. Robinault, PhD (1973)